PRO1 MICRO

Sistema de Control de Higiene

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PRO1 MICRO

Hygiene Monitoring System.

High sensitivity pen system for the detection and quantification of surface proteins and allergens.

Usage

Designed to detect and quantify proteins, allergens and reducing agents with high sensitivity on surfaces and difficult-to-access areas after the cleaning process.

Applicable Regulation

ISO 15883-1: 2006 / HTM 01-01: 2016 / HTM 01-05: 2013

Authorization

Designed under Quality Management System standards ISO 13485:2003/NS-EN ISO 13485:2012.

Characteristics

- The system has a high absorption swab which allows the collection of samples on different types of surfaces with the same efficiency, as well as a reaction solution.
- Pen diameter: 12,4 mm; Pen length: 163,6 mm; Swab length: 107,2 mm.
- Detection limit: 1 µg
- Sensitivity: 0.3 µg
- Qualitative analysis: When comparing the color obtained in the solution of the pencil with the reference table, a visual estimation of surface cleaning can be made.

OK - Clean, no further actions required.

Warning! - Re-clean.

x/xx - Re-clean and re-test.

•Quantitative analysis: Using the Bionova® IC10/20FR, Bionova® IC10/20FRLCD or Bionova® MiniPRO auto-reader incubators. Read the instructions for use of the device previously.

Environmental conditions during manufacture

15-30 °C (room temperature), RH 30-80 %.

Storage conditions

T = 2-30 °C, RH 30-80 %, store in dark place.

Transportation conditions

Storage conditions should be strictly followed.

Transport in closed and strengthened boxes in order to avoid damages. Transportation of this product does not represent a risk for health. Product can be kept at temperature below 25 °C for up to 25 days during transportation without modifying its shelf-life (provided that the subsequent storage conditions are respected)

Shelf-life

24 months when stored refrigerated (4°C) and 6 months when stored at room temperature (up to 25°C).

Packing

Boxes with 100 pencils + 1 swab moisturizer, boxes with 20 pencils + 1 swab moisturizer or foil bags with 8 pencils + 1 swab moisturizer (sample).

Packing information: product code and description, process for intended use, classification according to regulation, manufacturer information and data on label.

Labeling

On product's box: Product code and description, batch number and expiration date.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

Other important information

Read product's directions for use thoroughly before use.

Note: When necessary, limits for each product's parameters should be specified.

Description of identified and/or predictable risks which could represent a risk for the product

Do not reuse.

Do not open until its use.

Do not touch the swab with your hands. Wear gloves, surgical mask and other hygiene measures.

Do not freeze.

High concentrations of alkaline sanitizers may cause false negative results in some cases. Hydrogen peroxide based disinfectants may cause a false positive color change reaction.