BT221

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BT221

Self-Contained Biological Indicator.

Rapid Readout Fluorescence System.

Usage

Monitoring of Steam Sterilization processes.

Conditions: gravity-air displacement cycles at 132 °C – 135 °C.

Exclusive conditions for United States:

- Gravity cycles at 132 °C, 3/10 minutes.
- Gravity cycles at 135 °C, 3/10 minutes.

Applicable Regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.

ISO 11138-1:2017 and ISO 11138-3:2017; IRAM 37102-1:1999 and IRAM 37102-3:1999.

Classification

Class 1, according to risk.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-1.

Characteristics

Polypropylene tube: 50.4 mm high x 8.5 mm external diameter. Wall thickness: 0.5 mm.

Polypropylene cap: 10.5 mm external diameter, 16.4 mm high. Wall thickness: 0.9 mm.

Cap filter: medical grade paper, 60.0 g/m², 17.0 mm diameter.

Glass ampoule: 35.0 - 40.0 mm high. External diameter: 6.8 mm. Wall thickness: 0.2 – 0.3 mm.

Culture medium 0.5 – 0.7 ml, purple color.

Polypropylene microfiber on spore carrier, 17.0 mm diameter, 80.0 g/m².

Spores carrier: Filter Paper: 16.0 mm, 160.0 g/m²

≥ 10⁶ *Geobacillus stearothermophilus* ATCC 7953 spores per vial.

Final fluorescence reading is performed after 1 hour-incubation at 60 °C (sensitivity: ≥ 97 %).

An optional visual pH color change confirmation could be made after 48 hours of incubation. If sterilization process has not been successful, culture medium will change to a greenish color first, and then to yellow during incubation at 60 °C, thus showing the presence of living spores. If sterilization process is successful culture medium will remain purple after the incubation process.

7-day readout is optional and not intended to be routinely performed; it is an initial validation of the 1 hour-reading. Fluorescence results may be compared to the 7-day visual reading.

NOTE: if 7-day readout is performed, a humidified environment will be required to prevent medium from drying out.

D-Value: Not lower than 1.5 minutes at 121 °C. Another D-value is informed at 132°C and at 135°C.

Z-Value is informed.

Storage conditions

T = 10-30 °C, RH 30-80 %, keep in a dark place in its original box.

Transportation conditions

Storage conditions should be strictly followed.

Products should be transported in closed and reinforced boxes in order to avoid damages. Product transportation does not represent any risk for human health.

Shelf-life

2 years.

Packing

50 units per box.

Packing information: product code and description, process for intended use, presentation, regulation, bacterial strain, storage conditions, manufacturer information and data on pack's label.

Labelling

On product: 17.0 mm x 33.0 mm polypropylene label. 1.5 mm x 10.0 mm chemical indicator line, printed with Steam reactive ink (Color change to brown). Graph showing final fluorescence reading time, product code, batch number, process for intended use and bacterial strain printed in black.

On product's pack: product code and description, batch number, bacterial load, manufacture and expiration date, barcode and datamatrix code.

NOTE: manufacture date is calculated by subtracting 24 months to the expiration date.

Possible target markets

Healthcare and Industry.

Other important information

Incubation at 60 °C must be performed in Bionova® Auto-Reader Incubators.

Read product's directions for use thoroughly before use.

Description of identified and/or predictable risks which could represent a risk for the product

Do not store the product near sterilizing agents.

Do not expose this product to EO, Dry Heat, Radiation or any sterilization process other than Steam.

Discard biological indicators after use according to your country's healthcare and safety regulations. Positive biological indicators can be sterilized in a gravity-air displacement steam sterilizer at 121 ° C for 30 minutes, at 132 ° C for 15 minutes or 134 ° C for 10 minutes; or in a dynamic-air removal steam sterilizer at 132 ° C for 4 minutes or 135 ° C for 3 minutes.